..Hi-Laser 
Light for your life

   Your Medical **System**

- .. δ Laser
- .. γ Laser
- .. Ω Laser

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*In 2010, New Age Italia presents a line of laser equipment to physical therapy professionals with a single objective: to have **total control of the parameters that generate the therapeutic laser beam.***

The technological development has now made it possible for the therapist to set all the parameters required with the maximum freedom: power, time, frequency from 100Hz to 20KHz, and duty cycle from 20% to 100% (continuous).

*The absolute new feature of New Age laser equipment is the possibility to perform therapies using **3 lasers** simultaneously at different wavelengths: **808nm 980nm 1064nm**, all transmitted on the same optical fibre.*

This unique feature makes it possible to treat different tissues and depths at the same time, concretely reducing the treatment time.

The colour touchscreen display and the rotary encoder with pushbutton enable an easy and intuitive approach with the machine. The real-time display of the average power emitted, the energy transmitted, and the total energy of the treatment guarantees total control of the therapy, thus providing the best instrument in the therapist's hands.

New Age also makes available its consolidated experience in the physical therapy field with the numerous programmes resident on each device, allowing the therapist to be operative and effective right from the first applications, achieving the best results in the shortest time possible. The high power of the New Age LASER modules makes it possible to quickly reach the therapeutic energy doses, thus reducing the time required for each therapeutic session. With the conventional method, interchangeable spacers of different lengths are used, placed between the patient and the head of the handpiece, in order to obtain LASER spots of different diameters. The New Age R&D Department, instead, has developed an innovative handpiece equipped with a system of lenses able to focus and defocus the LASER beam.

.. **α** Laser

The spot goes from a minimum of 8mm to a maximum of 20mm diameter, all this by simply turning the ring nut on the handpiece. This eliminates the need for the fragile and inconvenient interchangeable components and also enables a more accurate regulation of the spot, which can therefore completely satisfy the needs of each individual patient. All this translates into the maximum freedom of action for the operator and therefore better results in a shorter time. In producing our equipment, New Age Italia has always made use of only the highest quality materials and avant-garde technologies, measures that guarantee the maximum efficiency and reliability to the therapist, who can use our products with the utmost confidence.

Biological effects of HI-LASER

Penetrating into the tissues, the LASER radiation causes various photochemical reactions at the cellular level that provide various beneficial biological effects.

Mitochondrial Stimulation: One of the most interesting effects brought about by the LASER is mitochondrial stimulation. Mitochondria are the energy generators of each cell and are responsible for the production of ATP (adenosine triphosphate), which constitutes an energy reserve of the cell. **ATP** is produced by **ADP** (adenosine diphosphate) plus **Pi** (phosphate group) plus **E** (energy), therefore: **ATP=ADP+P+E**.

The more ATP is present in a cell, the healthier and more energetic the cell. Studies have shown that cells stimulated by LASER light increase their production of ATP; therefore, when inflammatory or degenerative processes are present, the cells have sufficient energy to recover an optimal state of health more quickly.

Activation of the microcirculation: One of the effects deriving from the use of high power LASER is the striking vasoactive effect on the microcirculation. Activation of the microcirculation leads to greater oxygenation and nutritional supply as well as better drainage of catabolites from the tissues.

Activation of lymphatic peristalsis: LASER light accelerates lymphatic peristalsis, which facilitates the absorption of interstitial liquids and the reduction of oedemas of flogistic and post-traumatic origin.

Hyperpolarisation of the membranes of the nerve fibres: Experimental studies have shown that therapeutic LASER treatments cause hyperpolarisation of the membranes of the nerve fibres. This effect seems to be linked to the closing of the membrane channels for potassium, due to modifications of the surface lipoproteins. Hyperpolarisation raises the threshold of excitability of the pain receptors.

Transformation of prostaglandins into prostacyclin: Therapeutic LASER treatments stimulate the transformation of prostaglandins into prostacyclin PG12, which has an antiflogistic, antioedemigenic, and analgic action.

Areas of application

The most common pathologies treated with the LASER therapy are those affecting the locomotor apparatus and the skin. In particular, LASER therapy is successfully employed in the treatment of:

- Arthralgias of various nature, whether rheumatic or degenerative (inflammatory and degenerative arthropathies, epicondylitis, gonalgia with and without effusion, myositis, polyarthritis, arthritis, arthrosis, ischialgia, lumbago).
- General traumatology, joint sprains, tendonitis, tenosinovitis, muscle sprains, ecchymosis, bursitis, enthesitis, overload pathologies, muscle and tendon lesions, torn muscles and tendons, muscle contusions.
- Motor rehabilitation post-surgery or following forced inactivity.
- Dermatology, psoriasis, cicatrisation, bedsores, herpes zoster.
- Oedema, arteriopathies.
- Circulatory deficit.

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- Muscle sprains
- Ecchymosis
- Bursitis
- Enthesitis
- Overload pathologies
- Muscle and tendon lesions
- Torn muscles and tendons
- Muscle contusions
- Motor rehabilitation post-surgery or following forced Inactivity
- Psoriasis, cicatrisation, bedsores, herpes zoster
- Oedema, arteriopathies
- Circulatory deficit

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No. Output channels: 1

No. of programmes: 40

Handpiece: able to focus and defocus the laser beam from 8mm to 20mm diameter

"Alpha LASER"

- Three output wavelengths laser 7W at 808nm, 13W at 980nm, 10W at 1064nm

40 preset programs

"Beta LASER"

- Two output wavelengths laser 7W at 808nm, 13W at 980nm

30 preset programs

"Gamma LASER"

- One wavelength output laser 7W at 808nm

14 preset programs

"Delta LASER"

- One wavelength output laser 13W at 980nm

16 preset programs

"Omega LASER"

- One wavelength output laser 14W at 1064nm

10 preset programs

For each model it is possible to adjust duty cycle from 20% to 100%, frequency from 100Hz to 20 KHz, treatment time from 1 to 60 minutes.

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STANDARD KIT

- Main device
- Laser treatment head
- Feeding cable
- User's manual

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NEW AGE HAS BEEN MARKED ACCORDING TO THE DIRECTIVE UNI EN ISO 9001-2008 AND UNI CEI EN 13485-2004
ALL PRODUCTS ARE MADE ACCORDING TO THE CEE 93/42 DIRECTIVE ON MEDICAL

New Age

www.newageitalia.it

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NewAge Italia Srl - Via Dè Brozzi, 3 - Lugo (RA) 48022 - Tel. +39.0545.32019